




OPERATION MODE & PRECAUTIONS FOR MICROS



REAGENTS CONTROL BLOOD CALIBRATION BLOOD



TEMPERATURE

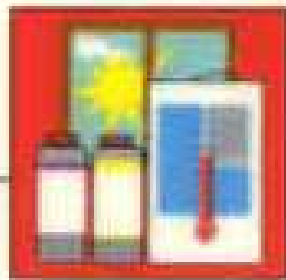

Reagents have to be stored at room Tpt (18°C to 25°C)
Control blood (4°C to 8°C)

Expiration date



TO CHECK !!

Do not use mixer !



Do not leave out of fridge more than 1 hour !!

BLOOD CONTROL THERMAL SHOCK



BLOOD MIXING



Rotative mixer or manual mixing between each sampling
30 minutes max !